F. BOSCH
MEDIZINTECHNIK

Model: **FB2405**

TENS + EMS İKİLİ TERAPİ
ELEKTRİKSEL STİMÜLATÖR

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EN User Manuel

Table of Contents

INDICATIONS AND CONTRAINDICATIONS	31
WARNINGS AND PRECAUTIONS	32
PACKAGE CONTENTS	38
ABOUT THE DEVICE	39
STEP BY STEP OPERATION GUIDE FOR TREATMENT	40
Preparing Skin for Running a Session	40
Inserting/Changing Batteries	41
Connecting the Cable to The Device	42
Turning ON the Device	42
Turning OFF the Device	43
Selecting the Treatment Time	43
Selecting the Program	44
Selecting the Therapy Intensity Level	46
Pre-programmed Treatments/Treatment Programmes	47
For a review of the placement of the electrodes / pads	50
SPECIAL FEATURES	52
CARE AND MAINTENANCE	53
TROUBLE SHOOTING	54
F. Bosch TENS+EMS STIMULATOR TECHNICAL SPECIFICATIONS	55
WARRANTY	55

Indications And Contraindications

READ THE OPERATION MANUAL BEFORE USING.

Read instruction manual before operation. Be sure to comply with all "CAUTIONS" and "WARNINGS" in the manual. Failure to follow instructions can cause harm to user or device.

Observe your physician's precise instructions and let them show you where to apply the electrodes. For Successful therapy, the correct application of the electrodes is an important factor.

INDICATIONS FOR USE

F Bosch TENS+EMS Dual Therapy System is a dual channel digital TENS/NMES electrical stimulator for active treatment application as the following intended use :

Transcutaneous Electrical Nerve Stimulator (TENS) can be used for the Following applications :

- For symptomatic relief of chronic intractable pain.

Neuromuscular electrical stimulation (NMS) stimulation can be used for the following applications :

- Relax muscle spasms
- Prevent or retard disuse atrophy
- Increase local blood circulation
- Re-educate muscles
- Maintain or increase the range of motion

Warnings And Precautions

WARNINGS

- If you are under the care of a Physician, consult with your Physician before using this System.
- The long-term effects of this System are not known.
- Do not place the pads on or close to your heart.
- Do not place the pads around or close to your neck. Do not apply stimulation over the neck. Severe spasm of the muscles may occur and the contractions may be strong enough to close the airway or cause difficulty in breathing. Stimulation over the neck could also have adverse effect hearing or blood pressure.
- Do not apply stimulation across the chest because the introduction of electrical current into the chest may cause rhythm disturbances to the heart.
- Do not place the pads on or around your head. The effects of stimulation of the brain are unknown.
- Do not use the pads over or close to sores.
- Do not place the pads on the front or sides of the neck across or through the heart (one pad on the front of the chest and one on the back), in the genital region, or on the head, because of the risk of stimulating inappropriate muscles and organs.
- Do not place the pads over any recent scars, broken or inflamed areas of infection or susceptibility to acne, thrombosis or other vascular problems (e.g. varicose veins), or any part of the body where feeling is limited.
- Do not place the pads over areas of injury or restricted movement (e.g. fractures or sprains).
- Do not use this System while sleeping.
- Do not use if you feel numbness.
- Do not use this System in or close to water.
- Do not apply stimulation across the chest because the introduction of electrical current into the chest may cause rhythm disturbances to the heart, which could be lethal.
- Do not use the pads over or close to cancerous lesions.
- Use the pads only on normal, healthy, clean and dry skin. Do not use the pads on open wounds or rashes, or over swollen, red, infected or inflamed skin.
- If you have ever had back surgery, consult your Physician before using this System.
- Electronic monitoring equipment (such as ECG and ECG alarms) may not operate properly when electrical stimulation is in use.
- You must position the pads and operation the unit ONLY as indicated in this manual.
- Avoid areas in injury or restricted movement (e.g. fractures or sprains)
- Avoid placing the pads over metal implants.

- Avoid using different size electrodes together can cause skin irritation or increased stimulation intensity under the smaller electrode. Some programs may require the use of different sized electrodes for treatment.
- Do not use in the bath or shower, or in an environment of elevated humidity (e.g. Sauna, hydrotherapy, etc).
- Operation in close proximity, such as 3 feet (1 meter), to shortwave or microwave therapy equipment may produce instability in the device output and may shut the device off.
- Do not use the device in an environment where flammable or explosive fumes may exist.
- Patient should never operate potentially dangerous machinery such as power saws, automobiles, etc. during electrical stimulation.

Wait before using next until :

- At least 6 weeks after the birth of your baby (you must consult your doctor before use).
- One month after an IUD contraceptive device (e.g. coil) has been fitted (you must consult your doctor before use).
- At least 3 months after having a caesarean section (you must consult your doctor before use).
- The heavy days of your period have finished, because vigorous abdominal exercise is not recommended at this time.

PRECAUTIONS

- Read User Manual before using this System for the first time.
- Keep this manual available whenever you use your System.
- The System is intended for personal use on healthy adult muscle only.
- The safety of using the System during pregnancy or birth has not been established.
- The effectiveness of the System depends greatly on a person's individual physical condition. It may not always be effective for every user.
- The safety of neuromuscular stimulation during pregnancy has not been established.

- Use caution when/if:
 - Sensory nerve damage is present by a loss of normal skin sensation.
 - Use caution prior to using this device on patients suspected of having heart disease.
 - Use caution for patients with suspected or diagnosed epilepsy when using this device.
 - Use caution following recent surgical procedures when muscle contraction may disrupt the healing process.
 - Use caution when there is a tendency to hemorrhage, such as following acute trauma or fracture.
 - A menstruating or pregnant uterus.
 - Patient experiences skin irritation due to electrical stimulation or the electrical conductive medium used, remove the electrodes, discontinue stimulation, and consult the clinician. Irritation may be reduced by an alternative conductive medium or an alternative electrode placement. Isolated cases of skin irritation may occur at the site of electrode placement following long term application.
- Place electrodes in accordance with illustrations in the User Manual.
- This unit should not be used while driving, operating machinery or during any activity in which involuntary muscle contractions may place the user at undue risk of injury.
- Some users may experience skin irritation or hypersensitivity due to the electrical stimulation or the conductive medium.
- Keep this device out of the reach of children. If the patient is a child, make sure he/she is properly supervised during electrical stimulation.
- Application of moderate heat (thermal wrap) to muscles as well as moistening skin prior to treatment improves treatment efficacy; use of cold packs on treated muscles after treatment is likewise recommended.
- This unit should only be used with the leads, electrodes and accessories provided by the manufacturer.
- The device is not intended for medical use, for the treatment of any medical condition or for any permanent physical changes.
- Contact Trimpeks, or an authorized dealer, if your unit is not working correctly. Do not use in the meantime.
- An effective session should not cause discomfort.
- For first time users, muscle stimulation can be an unusual sensation. We recommend that you begin in a seated position with low stimulation intensity settings to familiarize yourself with the sensation before progressing to higher intensity settings.
- The leads and pads must not be connected to other objects.
- Do not over exert yourself while using muscle stimulation. Any workout should be at a comfortable level for you.

- Do not place pads over jewelry or body piercings.

Use Caution and consult your Physician before using System if any of the following conditions apply to you:

- You have any serious illness or injury not mentioned in this guide.
- You have recently undergone a surgical procedure.
- You take insulin for diabetes.
- You use the unit as part of a rehabilitation program.
- If you have suspected or diagnosed heart problem.
- If you have suspected or diagnosed epilepsy.
- If you have a tendency to bleed internally following an injury.
- If you recently had surgery, or have ever had surgery on your back.
- If areas of skin lack normal sensations, such as skin that tingles or is numb.
- During menstruation or during pregnancy.
- Some people may feel skin irritation or experience a very sensitive feeling in the skin due to electrical stimulation. If this occurs, stop using your System and consult your Physician.
- If skin under one of more pads feels irritated after using the stimulator for a long period of time, use the stimulator for a shorter period of time.
- Minor redness at stimulation placement is a normal skin reaction. It is not considered as skin irritation, and it will normally disappear within 30 minutes after the electrodes are removed. If the redness does not disappear after 30 minutes from the removal of electrodes, do not use the stimulator again until after the excessive redness has disappeared.
- Turn off the stimulator if the stimulation feels unpleasant or does not provide pain relief.
- Keep your System out of the reach of children.
- Use your stimulator only with the pads, snap cables and accessories recommended by the manufacture.
- Do not use this System when driving, operating machinery or when swimming.
- Before removing the belt and pads, be sure to power off device to avoid unpleasant stimulation.

After strenuous exercises or exertion :

- Always use lower intensity to avoid muscle fatigue.

IMPORTANT:

- Effectiveness is highly dependent upon patient selection by a clinician qualified in the management of pain or rehabilitation.
- Do not use your unit at the same time as any other device, which transfers an electrical current into the body (e.g. another muscle stimulator).
- Cease using your unit if you are feeling light headed or faint. Consult doctor if this happens.
- Do not touch the pads or metal studs while the unit is switched on.
- Do not use unit if you are wearing a belly button ring. Remove ring before session.
- Use the device with only the leads and electrodes provided for use by the Trimpeks with your device. Any others may not be compatible with your unit and could degrade the minimum safety levels. Use only the electrode placements and stimulation settings prescribed by your practitioner.
- This device is for external use only.

Note: If you are in any doubt about using device for any reason, please consult your doctor before

PAD/ELECTRODE PRECAUTIONS

- To reposition the pads during a session, always pause the program currently running, reposition the pads and then restart the program again.
- The pads are for single person use only.
- Do not plunge the pads into water.
- Do not apply solvents of any kind to the pads.
- Always ensure the unit is OFF before removing the pads.
- Apply the whole surface of the pads firmly to the skin. Do not use pads, which do not adhere properly to the skin.
- If your skin is red under the pad after a session, do not start another session in the same area until your redness has completely disappeared.

Adverse Reactions


- You may experience skin irritation and burns beneath the stimulation electrodes applied to your skin.
- You may experience headache and other painful sensations during or following the application of electrical stimulation near your eyes and to your head and face.
- You should stop using the device and should consult with your physician if you experience adverse reactions from the device.

Conditions that may affect your F. Bosch TENS+EMS Dual Therapy System


Since the stimulator is a battery-operated electronic System, its output performance and safety may be affected greatly in extreme humidity. Therefore, it is very important to keep the stimulator dry to ensure the safety and performance of the stimulator.

Package Contents


1. F. Bosch TENS+EMS Dual Therapy System
2. 1 clip holder
3. 4 electrode pads
4. 3 AAA batteries
5. 2 lead wires
6. 1 Pouch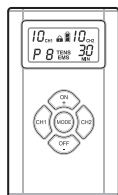


1

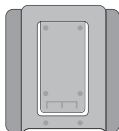
2

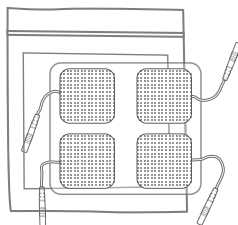
3

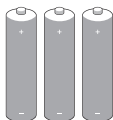4

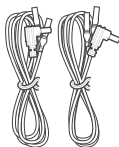
5

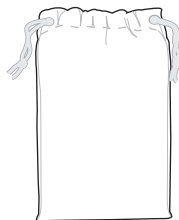6

About Device

1. Power on / adjust / increase setting key
2. Power off / adjust / decrease setting key
3. CH1 key
4. CH2 key
5. Therapy Time/Mode/Program selection Key
6. Program number
7. Therapy time remaining
8. CH1 intensity level
9. CH2 intensity level
10. Lock status indicator
11. Battery status indicator
12. Batteries compartment
13. Battery cover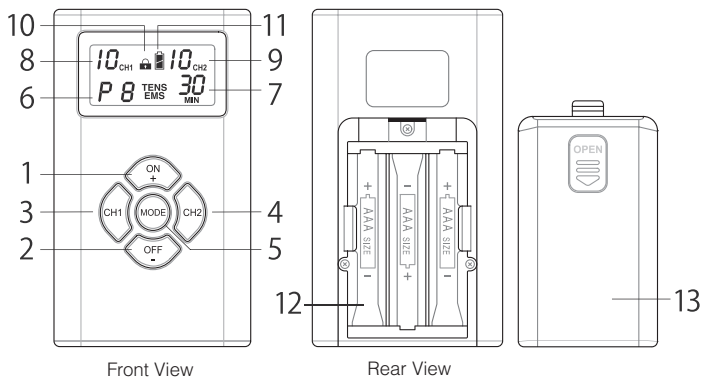

Step By Step Operation Guide For Treatment

Preparing The Skin For Running A Session


Proper preparation of the skin covered by the electrodes allows more stimulation to reach targeted tissues, prolongs electrode life, and reduces the risk of skin irritation. After connecting the lead wire(s) to the stimulator, use the following steps to prepare your skin at the electrode placement sites:

1. Determine the placement sites for the electrodes.
2. Wash the area with mild soap and water (do not use alcohol). Rinse and dry thoroughly.
3. Trim excess body hair from the area with scissors (do not shave).
4. Optionally, apply skin prep to the area to form a protective barrier on your skin. Apply, let dry, and apply electrode as directed. This will both reduce the chance of skin irritation and extend the life of your electrodes.
5. When removing electrodes, always remove by pulling in the direction of hair growth.
6. It may be helpful to apply skin lotion on electrode placement area when not wearing electrodes.

Please refer to the “Overview of the placement of the pads” on pages 50 and 51.

Inserting/Changing the Batteries

1. Open the battery compartment at the back of the device by pushing the battery cover labelled "Open" downward (this area features raised marks for easy identification).
2. Insert 3 AAA (1.5 V) batteries in the battery compartment; make sure to match up the symbols (+/-).
3. Close the battery cover by carefully placing the stud into the slot in the rear area and sliding it upward, applying slight pressure.
4. Follow the same procedure when replacing the battery at a later date.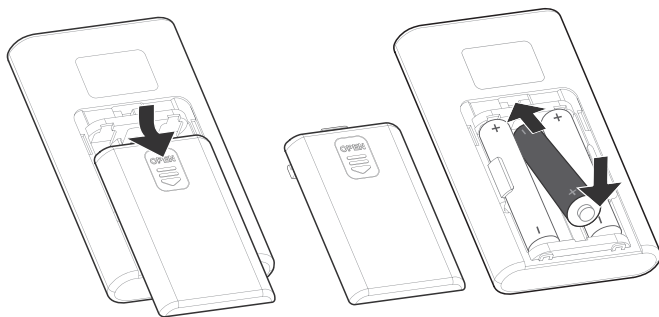


NOTE : for important precautions regarding the batteries, please be informed:

- Always use only 3x1.5V(AAA) batteries.
- Keep away from children.
- Do not recharge.
- Do not short-circuit.
- Do not throw into a fire.
- Please recycle. Do not dispose of old batteries with your household waste; dispose of them safely at your recycling centre or business where the batteries were purchased.

Connecting the Cable to The Device

Connecting the lead wire to the electrodes before applying to the Skin.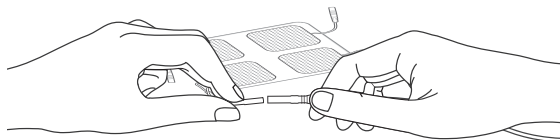


Turning On the Device

1. Press and hold the ON+ button for one (1) second to turn on the device.
2. The most recently selected treatment time and program will flash when the unit is turned on.

NOTE: Do not turn the unit on until all electrodes and lead wires are properly attached.


NOTE: When stimulating the muscles of the arms or legs bear in mind that the muscle contraction may cause involuntary limb movement which could cause injury to you or others. Always ensure the limb is secured to prevent movement.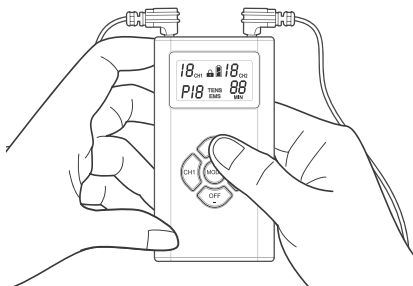

Turning Off the Device

1. The device turns off automatically after the therapy session time has elapsed.
2. To turn the unit off manually, press the OFF- button for three (3) seconds. The display will blank and the device will turn off.
3. In an emergency you may also pull the connector(s) from the device and then remove the belt.

NOTE: To prevent unpleasant electric shocks, never remove the device while it is still turned on.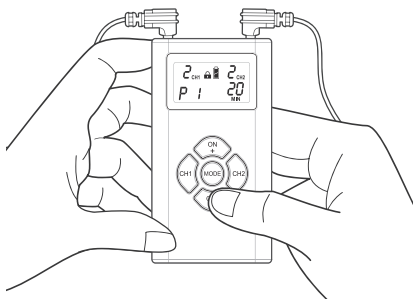

Selecting the Treatment Time

1. Press MODE. The preset (default) treatment time will flash on the display.
2. To increase or decrease the treatment time, press the button ON + (to increase) or the button OFF - (to decrease) repeatedly until the desired duration appears on the display.
3. Press MODE again to save your selection. The treatment time you selected will appear on the display the next time you turn the device on.

NOTE: If you change programs during the course of a therapy session, the treatment time will not reset unless you manually reset it by performing the steps described above.

Selecting the Program

The Device offers 8 different preset treatment programs respectively for TENS /NMES modes; the programs differ with respect to varying pulse widths and frequencies .

For TENS programs:

Program	Pulse width (μS)	Pulse rate (Hz)	Waveform Type
P1	260	15	Constant
P2	260	60	Burst
P3	260	60	Constant
P4	260~156	2~60	Dense-Disperse
P5	260~156	60	Modulation Pulse Width
P6	260	7~60	SD Pulse Rate
P7	260~156	60	SD Pulse Width
P8	Recycle(P1~P7)		

All electrical specification ±20%

For NMES programs :


	Pulse Width (μS)	Ramp up (sec)	Hold on (sec)	Ramp down (sec)	Off Time (sec)	Pulse rate (Hz)	Function Mode
P1	260 μS (fixed)	2	3	2	2	70	5
P2		2	4	2	3	60	5
P3		2	5	2	4	50	5
P4		2	6	2	5	50	5
P5		2	2	2	6	50	A
P6		2	4	2	8	60	A
P7		2	6	2	10	70	A
P8						7~60	C

All electrical specification ±20%

Selecting the Program


1. Press MODE after treatment time is set. The preset (default) therapy mode TENS/EMS will flash on the display. Use ON + (to increase) or the button OFF – (to decrease), if you would like to change the therapy mode.
2. Press Mode again, the numeric number of program is then flashing. Press the button ON + (to increase) or the button OFF – (to decrease) for choice of program of the selected modality.
3. Press MODE again to save your selection. The treatment time you selected will appear on the display the next time you turn the device on.

NOTE: If you change programs during the course of a therapy session, the treatment time will not reset unless you manually reset it by performing the steps described above.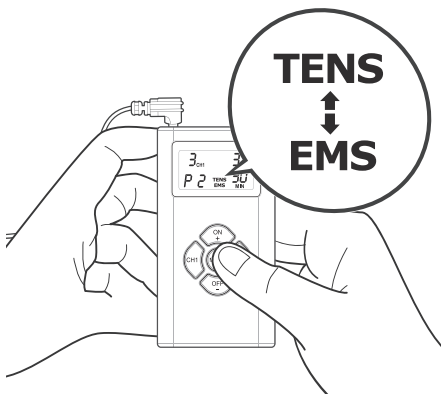

Selecting the Therapy Intensity Level


1. Intensity is adjustable according to the channel selected.
Select the channel you wish to adjust by pressing CH1 or CH2. "CH1" or "CH2" will flash on the display.
2. To increase or decrease the intensity, press ON + (to increase) or OFF - (to decrease) repeatedly until the desired intensity level flashes on the display.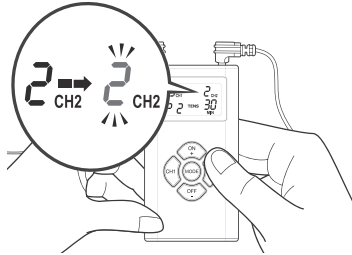

NOTE: You will feel the intensity increase or decrease as you select the intensity level. You can use this as a guide to select a level that is comfortable for you.

NOTE: If you change therapy mode/program during the course of a therapy session, the intensity level will reset to "0" showing on the screen, for safety reason.

3. Press MODE to save your selection.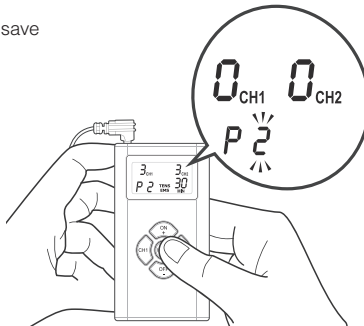

Pre-programmed Treatments/Treatment Programmes of the F. BOSCH TENS+EMS DUAL THERAPY SYSTEM in the “TENS” treatment mode

TENS (Transcutaneous Electrical Nerve Stimulation)

Treatment Program	Type of Pain	Treatment Objective	Description of Sensations
P1 / C mode	Chronic pain	Reduction in muscle pain	Continuous, comfortable tingling sensation. Pain should reduce gradually following treatment.
P2 / B mode	Acute pain	Pain block and help in relieving muscle spasms	Comfortable pulsating sensation. Pain should ease quickly during treatment.
P3 / C mode	Chronic pain	Pain block for muscle pain	Continuous pulsating sensation. Following treatment pain should be reduced and should gradually disappear.
P4 / D-D mode	Chronic pain	Increased release of endorphins and pain block	Wave-like alternation between tingling and pulsating sensations. Treatment should result in a marked reduction in pain.
P5 / MW mode	Chronic pain	Intensive loosening of tired muscles, massage	Variable, wave-like, pleasant gentle tingling.
P6/SDR	Chronic pain	Relaxation for over-exerted muscles, massage	Alternating, wave-like comfortable tingling and pressure.
P7 / SDW	Chronic pain	Prevention of habituation effect, Alternating massage	Varying intensive yet comfortable tingling and pressure.
P8 (Combination of Programmes P1 -P7)	Acute and chronic pain	Pain block and endorphin release	A combination of sensations produced by Programmes 1-7.


Pre-programmed Treatments/Treatment Programmes of the F. BOSCH TENS+EMS DUAL THERAPY SYSTEM in the “TENS” treatment mode

NMES (Neuromuscular Electrical Stimulation)

Treatment Program	What you should feel and advantages	Suggestion
P1 Training preparation	This is a gentle warm-up programme for the muscles. It feels like a rhythmical massage.	Gradually build up intensity until you have reached an intense but comfortable level of muscle movement. Duration: 10 minutes.
P2 Muscle strengthening	This programme uses a frequency that allows the muscle fibres to pulsate. This leads to an increase in the muscles oxygen capacity and is used to improve on maximum muscle strength.	This programme incorporates a series of active phases followed by short relaxation periods. Gradually increase the level of intensity until you achieve a strong and deep contraction. However, do not continue to increase beyond comfort level. Duration: 20 minutes.
P3 Muscle strengthening	This programme uses a pulsating frequency that massages the muscle fibres. This improves the muscles oxidative capacity to increase maximum muscle strength.	This programme incorporates a series of active phases followed by short relaxation periods. Gradually increase the level of intensity until you achieve a strong and deep contraction. However, do not continue to increase beyond comfort level. Duration: 20 minutes.
P4 Muscle strengthening	This programme uses a pulsating frequency that massages the muscle fibres. This improves the muscles oxidative capacity to increase maximum muscle strength.	This programme incorporates a series of active phases followed by short relaxation periods. Gradually increase the level of intensity until you achieve a strong and deep contraction. However, do not continue to increase beyond comfort level. Duration: 20 minutes.


Treatment Program	What you should feel and advantages	Suggestion
P5 Muscle strengthening	This programme uses a frequency that allows the muscle fibres to pulsate. The channels work in pairs alternating between the different sides. This improves the muscles oxidative capacity to increase maximum muscle strength.	This programme incorporates a series of active phases followed by short relaxation periods. Gradually increase the level of intensity until you achieve a strong and deep contraction. However, do not continue to increase beyond comfort level. Duration: 20 minutes.
P6 Muscle strengthening	This programme uses a pulsating frequency that massages the muscle fibres. The channels work in pairs alternating between the different sides. This improves the muscles oxidative capacity to increase maximum muscle strength.	This consists of a series of working phases followed by short relaxation periods. Gradually increase the level of intensity until you achieve a strong and deep contraction. However, do not continue to increase beyond comfort level. Duration: 20 minutes.
P7 Active recovery	This programme activates muscles in short contraction/relaxation cycles. It feels like a kneading massage action. Here the channels work in pairs alternating between the two sides.	Use this programme following an intense training session to promote recovery and relaxation.
P8 Building stamina	This programme uses a sequence of low-frequency pulsations to develop the oxygen-dependent capacity and supply capillary vessels. This benefits slowly twitching muscle fibres. Resistance to fatigue is increased during long training sessions of moderate intensity.	This programme incorporates an alternating series of active phases followed by short relaxation periods, lasting several seconds. Gradually increase the level of intensity until you achieve a strong and deep contraction. However, do not continue to increase beyond comfort level. Duration: 20 minutes.

For a review of the placement of the electrodes / pads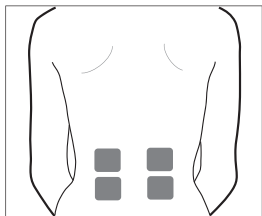


LOWER BACK

Place a pair of pads horizontally either side of your spine on the lower part of the back.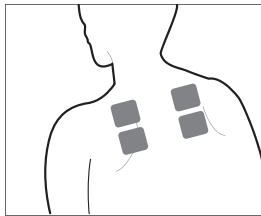


UPPER BACK

Place a pair of pads horizontally either side of your spine on the upper part of the back.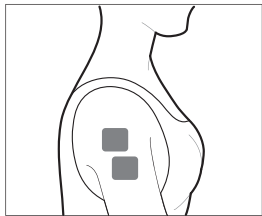


SHOULDER

Place one half of the pad on the front of your shoulder and the other on the side.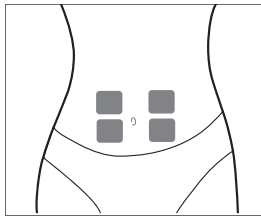


ABS

Place each pair of pads vertically either side of your navel.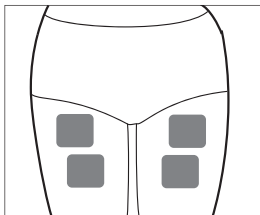


FRONT OF THIGHS

Place each pair of pads horizontally across each thigh muscle.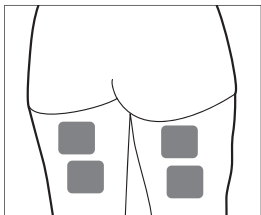


BACK OF THIGHS

Place each pair of pads horizontally across your hamstrings.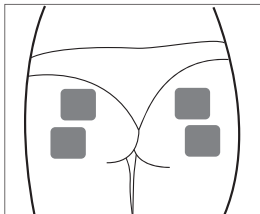

BOTTOM

Place the pads horizontally across the buttocks halfway between the midline and the side of your body

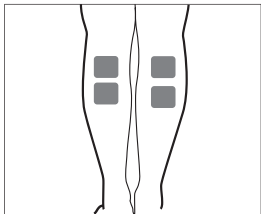CALF MUSCLES

Place one pair of pads horizontally across each calf muscle. Do not place them too low on the leg, as this can result in an uncomfortable contraction.

Special Features

TREATMENT TIME

The device offers 12 preset times: 5, 10, 15, 20, 25, 30, 35, 40, 45, 50, 55 and 60 minutes.

Time will countdown on the display in 1-minute increments for the duration of your session.

- The device turns off automatically when the therapy time has elapsed.
- The most recently set therapy time is stored.
- If you alter the program mode during your therapy, the therapy time won't restart, unless you reset the therapy time.
- The last treatment program you used will appear on the display, when you turn on the device.
- To change the program, press ON + or OFF – repeatedly until the desired program appears on the display.
- Press MODE to save your selection. The program you selected will appear on the display the next time you turn on the program.

LOCK FUNCTION

Press and hold the ON + and OFF – keys simultaneously for 1 second to lock/unlock the device.

AUTOMATIC SHUT OFF

- The device automatically turns off when no button is pressed for 60 seconds.
- The device automatically turns off when the time for your therapy session has elapsed.

INTENSITY LEVEL RESET

For your safety and comfort, the intensity level will reset to "0" each time the device turns off, including after therapy sessions.

The treatment will discontinue if the electrodes are not properly placed well, and/or any entry for changing the mode setting during therapy session, it shall initiate to lowest intensity level, showing "0" on the screen.

LOW BATTERY STATUS INDICATOR

The battery status indicator will light whenever the battery is low. This means that soon you have to replace the batteries.

Care And Maintenance

STIMULATOR

The stimulator may be wiped clean with a small amount of soapy water on a clean cloth. Do not submerge the stimulator in liquids or expose it to large amounts of water.

- Never use aggressive cleaning products or stiff brushes to clean the device.
- Remove the battery before cleaning the device.
- Do not use the device again until it is completely dry.
- Do not expose the device to direct sunlight and protect it from dirt and moisture.

CABLES

- Disconnect the cables from the stimulator and electrodes.
- Do not pull on the cables, but on the connectors attached to the ends of the cables.
- Store the stimulator with the cables in a clean, dry place.

ELECTRODE

The electrode pads are disposable and use an adhesive that will dry after prolonged usage or storage. Pads should be replaced when they lose their adhesive quality, or you sense a change in stimulation sensation.

If you're in doubt about the integrity of the pads, order fresh pads please order online at **www.trimpeks.com**, or contact authorized distributor(s).

HOW TO STORE YOUR SYSTEM

1. Store your System at room temperature in a dry place, out of the reach of children.
2. If the stimulator will not be used for more than a week, remove the battery from the stimulator.

Trouble Shooting

Always check the unit and accessories before use to prevent damage and defects; these are some of the simple checks:

1. Make sure the battery has sufficient charge and is not corroded.
2. Make sure the cables fit tightly into the connection sockets of the device. The table below shows some common defects. If you cannot remedy the defects as described, contact your unit provider if it is not possible to remedy in the manner described.

Defect	Cause	Remedy
The device does not turn on	No battery or bad battery	Replace battery
The device turns on and then off again	Battery not inserted properly	Insert battery again Replace battery
	Battery life expired	Replace battery
The device turns on, but does not generate electric pulses	Cable broken	Replace cable
	Cable not connected Properly	Connect cable properly
	Treatment time has Expired	Switch unit to the OFF Position and switch back on.
The unit does not turn on even though new batteries have been inserted		Call Customer Service

F. BOSCH TENS+EMS Stimulator Technical Specifications

Channel	: Dual, isolated between channels.
Pulse Amplitude	: Adjustable 0 – 80mA peak into 500 Ω load each channel.
Pulse Rate	: As pre-programming operation mode.
Pulse Width	: As pre-programming operation mode.
Software ramp up feature	: Pulse width ramp up when change mode.
Timer	: 5~60 min. selectable.
LCD	: Show modes, pulse rate, pulse width, timer, CH1/CH2, intensity level.
Wave Form	: Symmetrical Bi-Phasic square pulse.
Max Charge per Pulse	: 20.8 micro-coulombs maximum.
Interdependency	: 5% maximum.
Power Source	: 3 x AAA / 4.5 Volt batteries

All electrical specifications are $\pm 20\%$ at 500 Ω load.

Warranty

This F BOSCH TENS+EMS DUAL THERAPY System carries a two years warranty from the date of purchase.

The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alterations or disassembly by unauthorized individuals.

The warranty applies to the main device and necessary parts and labor relating thereto. Battery, lead wires, electrodes, and other accessories are warranted to be free from defects in workmanship and materials at the time of delivery.

The distributor reserves the right to replace or repair the unit at their discretion.

EXPLANATION OF SYMBOLS ON UNIT

	CE mark
	Manufacturer
	Read the instructions carefully before using this device
	Waste Electrical and Electronic Equipment Directive
	Type BF equipment
	Lot Barcode

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**For the probable changes of the technical
services, please check our below website.
www.trimpeks.com*

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SAN. VE TİC. LTD. ŞTİ.**